


CYCLO G6™ Quick Setup Guide*

STEP 1

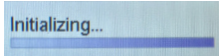

Turn key to “On.” Wait for the initialization to complete.



STEP 2



Plug in probe.

STEP 3



Press “OK” on defaults screen.

STEP 4



Press “OK” on Safety Warning Screen.

STEP 5



Press “Preset.”


Note: Please refer to the Cyclo G6 Operator Manual to create a new preset.

STEP 6



Press desired preset.

STEP 7

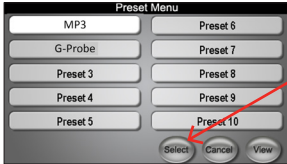


Press “Select”.

STEP 8



Press “Treat” button. “Treat” button will turn a constant green and laser is ready.

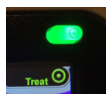

Parameters shown here are for illustration only.

Treatment parameters are solely at the discretion of the treating physician.


RESETTING COUNTDOWN TIMER



Press here.

CYCLO G6¹ Quick Setup Guide*

Not a substitute for the Operator's Manual.

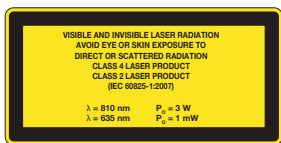

CE 0086



Emergo Europe
Molenstraat 15, 2513 BH, The Hague, The Netherlands
Tel.: (31) (0) 70 345-8570, Fax: (31) (0) 70 346-7299



IRIDEX

1212 Terra Bella Avenue
Mountain View, CA 94043 USA
800.388.4747 (U.S. inquiries)
info@iridex.com (U.S. & int'l inquiries)
www.iridex.com



WEEE Guidance. Contact IRIDEX or your distributor for disposal information.

*Please reference the CYCLO G6 Operator's Manual for more information on maintenance, safety precautions and any cleaning, disinfecting instructions.